



Guide to IntegradWeb Keyboard Shortcuts

IntegradWeb Keyboard Shortcuts

Action with system-supplied keyboard shortcut

Page/scroll the key image page -----	Use the control key while clicking the mouse or arrow keys
Pan while in Zoom mode -----	Use the middle mouse key roller
Zoom while in Pan mode -----	Use the middle mouse key roller
Open presets while in W/L mode -----	Use the middle mouse key roller
Get 1.00 zoom -----	Use the shift key
Refresh the study list -----	F5

Action with user-configured keyboard shortcut

Command	Key	Command	Key
Jump to first image	home	Window/Level mode	W
Jump to last image	end	W/L editor	ctrl+alt+W
Jump to middle image	insert	Invert	V
Page fast	page↑/page↓	Mirror	9
Patient Folder	F1	Sharpen	S
Triangulation	F2	Rotate	0
Update the study	F11	Zoom mode	Z
Cine	C	Set zoom to 1	1
Done	D or F12	Set zoom to 1.5	5
Exit	X	Set zoom to 2	2
Flip	F	Set zoom to 2.5	ctrl+2
Magnifying Glass	G	Set zoom to 3	3
Insert key image	I	Set zoom to 3.5	ctrl+3
Key image preview	K	Set zoom to 4	4
Labels (annotations)	L	Undo the zoom	6
Measurements	M	Next preset window	space bar
Page/Scroll	P	First page	home
Reset auto windowing	R	Last page	end
Recommended W-C	Q	Next page	up arrow
Fast backward	shift+left or shift+↓	Previous page	down arrow
Fast forward	shift+right or shift+↑	Page jump up	page ↑
		Page jump down	page ↓
		Edit key image temp.	ctrl+alt+P